

PRIMEXPROTECT



PRIMEXPROTECT'S APPLICATION

ASSEMBLY INSTRUCTIONS

DISINFECT PRIOR TO USE

1

REMOVE FILM FROM SHIELD.

2

PEEL FILM FROM FOAM

Peel the film from the adhesive on the foam.

3

ADHERE THE FOAM TO THE SHIELD.

Line the foam piece to the marking on the shield and attach to the plastic. Apply with pressure.

Clean area with isopropyl alcohol for better adhesion

4

VELCRO

Use the velcro straps to adjust the fit of the shield on the head.

CLEANING INSTRUCTIONS



To avoid damage to your PrimexProtect face shield, we recommend cleaning with isopropyl alcohol or a soap solution using any gentle dish soap.

PRIMEXPROTECT



RESISTANT TO SOLIDS & LIQUIDS

ASSEMBLY INSTRUCTIONS

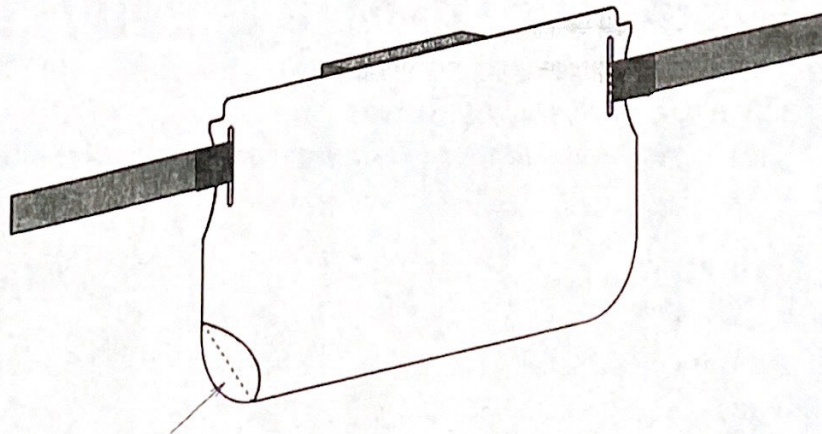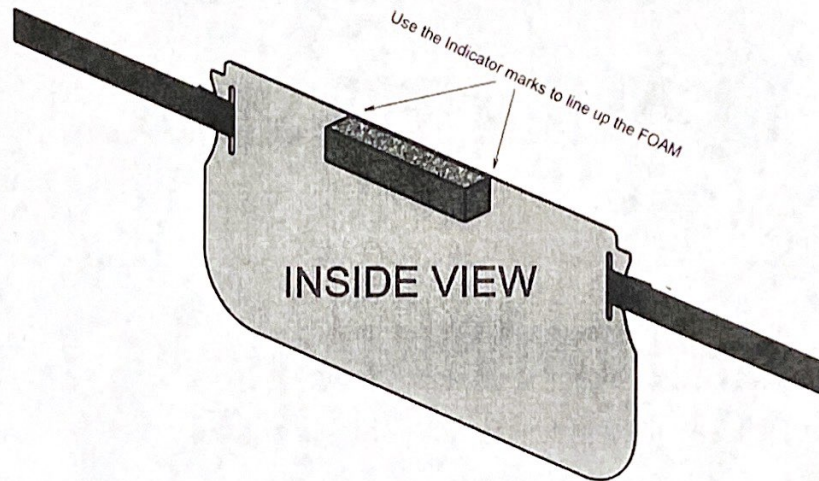

Remove Film from Shield

PRIMEX 
DESIGN & FABRICATION

A subsidiary of Primex Plastics Corporation